Repair Service Phaco & Harmonic Scalpels

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The Phaco Emulsificator Hand Pieces operates on the principle of the Piezoelectric Effect. When crystals are subjected to an electrical field, they expand and contract in direct proportion to that electrical field converting electrical energy into mechanical energy. In general, a handpiece will break down after approximately 500 procedures because of crystals cracked, cord wire damage, connector damage, tip damage, jammed needles, clogged aspiration port...

We repair and refurbish Alcon 20000 Legacy, AMO Sovereign, Bausch & Lomb Millennium, Bausch & Lomb Premiere, OMS Series 3, OMS Series 4, Chiron Catalyst, Alcon 10000, Alcon Legacy NeoSoniX & Alcon Infiniti Ultrasonic hand pieces. Please ask for our Cross Reference Chart for more information.

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Disposal of waste medical electrical and electronic equipments by users in the European Union. This symbol on the product or on its packaging indicates that this product must not be disposed of with your other kind of waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

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Harmonic Scalpels

Escalpelos armónicos

Harmonic Scalpels are ultrasonic instruments that cut and cauterize at the same time, coagulate blood proteins, at 55,000 Hz and operate at a much lower temperature than traditional surgical units. Harmonic Scalpels are used in all types of open and laparoscopic procedures and are very similar to phaco handpiece in technology. The scalpel has all of the same type components as the Phaco handpiece: four piezoelectric crystals, connector, cable, electrodes, o-rings and seals. Repair procedure mimics that of the phacos: complete disassembly of scalpel, discard cable, connector, crystals, electrodes and o-rings. We offer repair of all 3 generations of Ethicon harmonic scalpel and the new Blue hand piece.

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Carril is the first spanish company to obtain the ISO 13485 certification for the repair of medical US and toco transducers.

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